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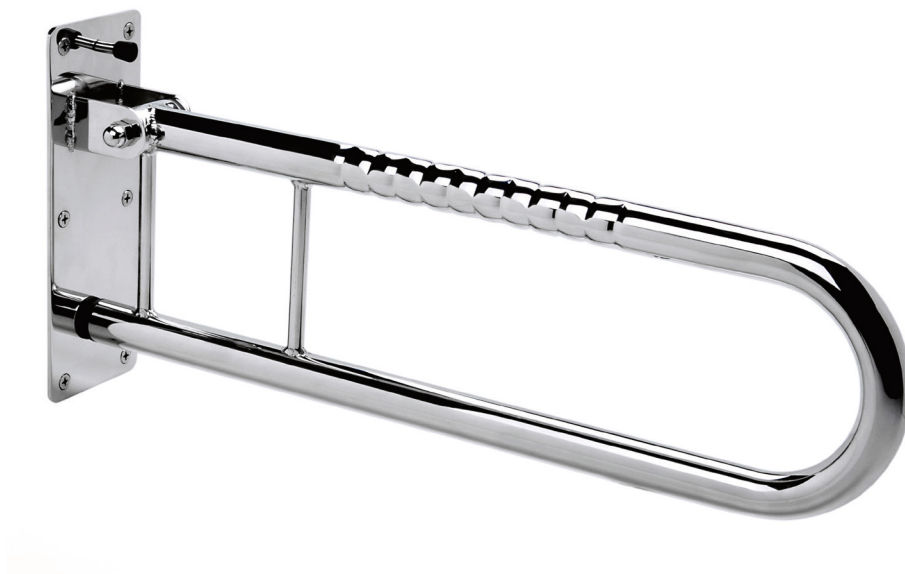
Table of contents

Preface	6
Declaration of conformity	6
Service life	6
Disposal	6
Identification plate	6
Reuse	6
Warranty	6
Parts of the product	7
Indication	7
Scope of delivery	7
Safety instructions	7
Assembly	7
Handling	7
Cleaning	7
Technical data	7

Preface

Thank you for choosing a product from Drive DeVilbiss.

You will not be disappointed by the design, functionality and quality of this product.

Please read these operating instructions carefully prior to initial use. They contain important safety information and valuable tips for the proper use and care. If you have questions or need additional information, contact your Drive DeVilbiss specialist dealer who supplied your product.

These operating instructions contain all of the important information for adjustment and operation. The most current operating instructions are available as a PDF online or can be requested from customer service.

Enlarged formats for visually impaired persons can be displayed. Blind patients must be instructed by their attendant. Repairs as well as some adjustments require special technical training and must therefore be made by the medical supplies dealer.

Declaration of conformity

For the product described, DRIVE MEDICAL GMBH & Co. KG declares compliance according to the medical device regulation (EU) 2017/745.

Service life

Our company assumes a service life of five years for this product as long as the product is used as intended and all maintenance and service requirements are observed.

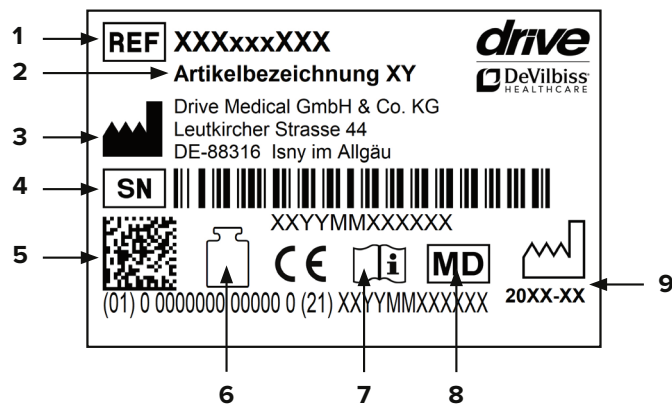
This service life may be significantly exceeded if the product is carefully handled, repaired, maintained and used and there are no technical limits based on the further development of science and technology.

The service life may also be significantly shortened through extreme and improper use, however. Determination of the service life by our company does not represent any additional guarantee.

Disposal

If you no longer need the product, please contact your specialist dealer or bring the product to your local disposal center.

Identification plate

1 Article number | 2 Article name | 3 Manufacturer | 4 Serial number | 5 UDI code | 6 Max. load | 7 Follow the instructions for use | 8 Medical device | 9 Production date (year-month)

Reuse

This product can be reused.

Please note that all technical documents needed for safe handling must also be given to the new user.

The product must be checked by the specialist dealer, cleaned, and restored to optimum condition.

Always follow the manufacturer's instructions during preparation; these are made available on request.

Accessories are available from your specialist dealer or at www.drivedevilbiss.de

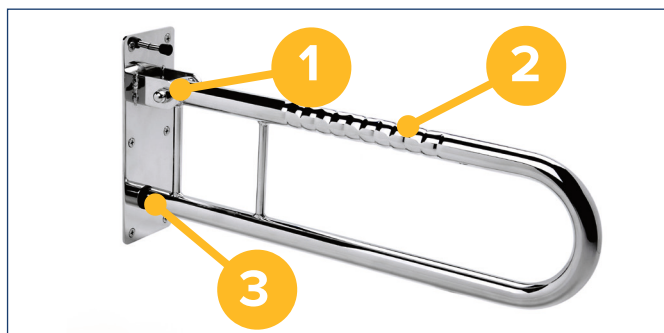
Warranty

The warranty period for the product supplied by us is 24 months from the date of purchase. Defective parts will be replaced free of charge if it can be shown that there is a defect in materials or manufacturing.

Parts subject to wear are generally not covered during the normal guarantee period unless the parts need repair or replacement which became necessary as a clear, direct consequence of a manufacturing or material error.

We reserve the right of error and modifications on the basis of technical improvements and changes in design.

Parts of the product

1. Clamping function screw connection
2. Stainless steel pivoting support handle
3. Fall guard

Indication

INDICATION:

Holding and support element made of stainless steel for the bath and toilet area. The support handle is mounted to the bathroom wall and facilitates or supports sitting down and standing up and enables safe and stable support of body weight.

CONTRAINDICTION:

Unsuitable for use if persons are not able to sit down and stand up by themselves.

Scope of delivery

1 x	Pivoting support handle
1 x	Plastic dowel
6 x	Stainless steel screws Ø 6 x 60
1 x	Instructions for use

Safety instructions

- Take note of the permitted maximum weight of the user.
- When pivoting upwards, clothing and body parts can be pinched.
- Protect the product from direct sunlight
- Prior to each use, ensure that the handle is intact and all parts are properly secured.
- Do not sit on the pivoting support handle!
- Check the screws regularly and tighten, if necessary.
- Ensure that your hands are dry.

Assembly

For safety reasons, assembly must be performed by an installation specialist. When attaching the product, the mounting height according to DIN 18024/25 must be observed. Only use mounting material which is approved for the structure. After assembly, test the secure hold as well as the pivoting and clamping function! Adjust the

clamping function using the screw connection (**photo 1**), if necessary. If the clamping function is inadequate, there is a risk that the handle will pivot downwards unintentionally. The standard mounting supplied contains a plastic dowel as well as stainless steel screws Ø 6 x 60 according to DIN 7995. Since these

are suitable only for mounting to the wall types concrete B 25, natural stone (dense structure), solid brick (M20) and sand-lime brick (Ks 12), the composition of the wall must be checked by a specialist for suitability for attaching the article. For other wall types, a company specialising in fastening technology must be consulted.

Handling

Pivot the support handle up or down as far as it will go, as needed.

Do not grip in the clamping area between the wall bracket and the pivoting support handle – risk of crushing!

Cleaning

- Clean regularly with a moist cloth!
- Do not use any aggressive cleaning agents!
- Do not use abrasive cleaners!
- Do not use any high-dose chlorine and vinegar-based cleaners!
- To disinfect, a spray disinfectant can be used such as Descosept AF or Bacillol AF.
- Check the screws regularly and tighten, if necessary.
- The product should not be exposed to temperatures above 60°C for long periods of time (for example, use in a sauna, etc.).
- In the event of defects, contact your specialist dealer from whom you purchased this product.

Technical data

Product	Length 85 cm	Length 60 cm
Article number	10495-20	10495-30
HMV (medical aids register) no.	04.40.05.1224	04.40.05.1224
Overall length	85 cm	60 cm
Mounting plate width	10 cm	10 cm
Mounting plate length	27 cm	27 cm
Tube diameter	3.2 cm	3.2 cm
Total weight	3.4 kg	2.9 kg
Max. load	120 kg	120 kg
Material	Stainless steel	Stainless steel

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Drive Medical GmbH & Co. KG